



eye **CRE-AT**  
*Vacuum Holder 4 Pigs*



Cleaning

# Vacuum Holder 4 Pigs

1. Vacuum Holder 4 Pigs
2. Remove the grid gently
3. Soak the Vacuum Holder 4 Pigs in soap water
4. Pull and push the syringe to fill the Vacuum Holder 4 Pigs with soap water and do the same in water to rinse the soap out



1



2



3



4

# Vacuum Holder 4 Pigs in pad

1. Vacuum Holder 4 Pigs in a pad
2. Remove the grid gently
3. Soak the Vacuum Holder 4 Pigs in pad in soap water
4. Pull and push the syringe to fill the Vacuum Holder 4 Pigs in pad with soap water and do the same in water to rinse the soap out.



1



2



3



4